

# Patient Identification Card



MRI  
Conditional

## I Have a Coloplast Prosthetic Implant.

M1782N 06.08  
70002067

To Whom it May Concern: In case of emergency, please be aware that I have a 3-piece Coloplast Inflatable Penile Implant that contains metal. The implant is MRI conditional. See opposite side and Instructions for Use for details. Cust. Serv.: 1-800-328-3863

Name: \_\_\_\_\_

Address: \_\_\_\_\_

City/State: \_\_\_\_\_

Surgeon: Dr. \_\_\_\_\_ Dr. Telephone: \_\_\_\_\_

Hospital: \_\_\_\_\_ City: \_\_\_\_\_

Date Implanted: \_\_\_\_\_

Health Insurance Provider: \_\_\_\_\_

Policy #: \_\_\_\_\_

In Case of Emergency, Please Notify:

Name: \_\_\_\_\_

Address: \_\_\_\_\_

City/State: \_\_\_\_\_

Phone: \_\_\_\_\_ Relationship: \_\_\_\_\_

## Titan<sup>®</sup> OTR Inflatable Penile Prosthesis




**Titan**  
**Coloplast**

# Instructions

## To Simulate an Erection


To transfer the fluid to the cylinders, locate the pump in the scrotum. Squeeze the bulb firmly a few times between the thumb and fingers until an erection is achieved.

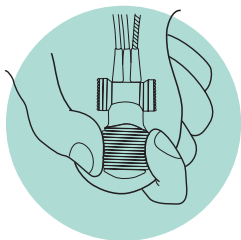

Squeeze bulb to inflate.

## To Return to a Flaccid State

Fluid pressure in the cylinders is released when the patient depresses the deflate valve buttons to enable fluid to return to the reservoir, thus returning the penis to the flaccid state.

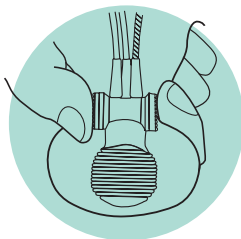

Momentarily depress the deflate valve buttons.

## To achieve a Flaccid State more quickly

Squeeze the penis gently with the other hand after depressing the deflate valve buttons.

If any problems are encountered during the use of this device, or any pain or discomfort are experienced, please contact your doctor immediately.



MRI  
Conditional

- Static magnetic field of 3 Tesla or less
- Spatial gradient field of 720 Gauss/cm or less.
- Maximum whole-body-averaged specification absorption rate (SAR) of 3W/kg for 15 minutes of scanning produced a temperature rise of less than 0.5°C.



Grasp and squeeze.